




# Dosimeter

## UVC-DOS-25

## Make sure your UV-CLEAN device is disinfecting

Now that you have decided to automate your disinfection regiment, you may be required to do regular quality control on your UVC device. Our Dosimeter testing cards offer a simple way to ensure the efficacy of your device. Once your device runs, you will quickly notice a color change confirming UVC light is being dispensed. Packs of 25 [UVC-DOS-25] are available through your local UV-CLEAN dealer.

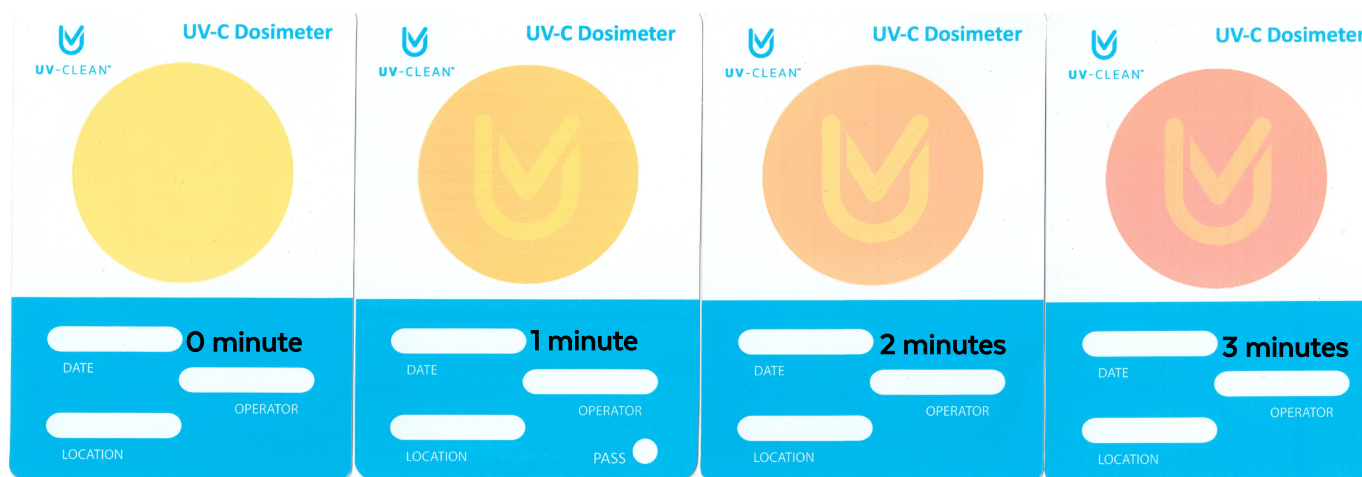

### Instructions For Use:

For best results place the card in a single position and allow a cleaning cycle to complete.

- 1) Record test information on the front of the card.
- 2) Locate and remove the adhesive strip.  
Place the card no more than 4 inches (10 cm) below UV-C light source. Card can be placed in a horizontal or vertical position. Always make sure the yellow dot faces toward the UV-C device so that a color reaction may occur.
- 3) Step away from the UV-C area and allow a full disinfection cycle to run (minimum 3 minutes).
- 4) Retrieve the exposed card. A visible yellow "UV" logo within darker surrounding color indicates a positive test  
Always check the card reading as soon as possible after the disinfection cycle and at latest within 24 hours after disinfection cycle to get most accurate result.
- 5) Discard card after use. Fading begins after 24 hours.  
The UV-C dosimeter cards should not be used for more than one test.

